



Instructions for Use

Intended Use: Flusso™ TFI allows for temporary interruption of gas flow to facilitate circuit disconnection.

Indications for Use: Patients requiring means of mechanical ventilation requiring circuit disconnection

Directions for Use

- Remove Flusso TFI from its package.
- Attach Flusso TFI to the endotracheal tube of the patient, then attach the ventilator circuit.
- Ensure all connections are secure. 15mm ID/15mm OD connections, device length is 5.65 cm.
- With your index finger and thumb activate (compress) the tabs equally on the device. Compress the tabs to the point where the tabs are flush with the sides of the device.
- Only the hand/fingers of a trained clinician should be used to activate Flusso TFI
- Inspect the device on a routine basis to ensure proper use and function. Should the device become soiled or damaged, do not use, discard and replace with a new one.
- Discard the adapter when replacing the circuit according to facility guidelines with a maximum usage of 7 days.



Caution / Attention:

- Do not use any devices to try and activate, lock or hold the compression tabs of Flusso TFI.
- The activation of Flusso TFI should not exceed 3 seconds or as per hospital protocol.
- Use only enough force to active the compression tabs failure to do so may allow gas flow to pass through.
- The device is intended for sale or on the order of a physician.

Contraindications:

- Patients who cannot tolerate 1.8 cc of dead space

Rx Only   **REF** 420-1979
- Non-Sterile
- Not made with
Natural Rubber Latex